



Passport for TravelScoot mobility aid following IATA regulations

Key informations for the Operator Approval


Type of mobility aid:

- **battery-powered** mobility aid
- **manufacturer = TravelScoot** (www.travelscoot.com) E-Mail: international@travelscoot.com
- models are TravelScoot Escape, TravelScoot Deluxe, TravelScoot Shopper, TravelScoot Shopping cruiser – all models are suitable for air traveling according to IATA regulations

Battery typ:

- Removable lithium (Li-Ion) battery (dry non-spillable)
- UN38.3 certified
- 252 Wh (Watt-hour) with 1,52 kg or older variations with 274 Wh, 283 Wh, 152 Wh.

Remove battery: switch to remove the battery (push both connector sides and gentle pull)

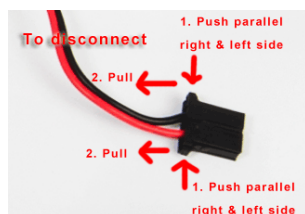

- must be removed from the mobility aid & must be packed in a strong and rigid packing
- electrical circuits (connector) needs to be isolated

Weight of the complete system: **15 kg** incl battery – **13,5 kg** without battery

Dimensions:

- Typ TravelScoot Escape & Shopping Cruiser: **94 x 53,5 x 70 cm** * (L x W x H)
- Typ TravelScoot Deluxe & Shopper **94 x 58,5 x 69 cm** (L x W x H) *

* in its air travel configuration at its lowest height (when all detachable parts are removed, and the mobility aid is ready to be loaded into the cargo compartment of the aircraft)

Preparation for loading:

- Remove all accessories like mirrors, baskets
- Back rest needs to be folded down or removed
 - (turn 180 degree – use little jumper below the seat where the back rest is fixed)
- Lower the handlebars as far as possible using the quick-release lever on the steering column
- Unit is permanent in freewheel mode – operator might use the parking brake

Squeeze the brake lever as far as possible (a) with one hand, depress the parking brake lever (b) with the other hand, and release the brake lever. To release the parking brake, squeeze the brake lever (c). The parking brake lever will release automatically.

For further questions you find the manual at:

<https://www.travelscoot.com/assets/pdf/travelscoot-shopping-cruiser-escape-manual-en.pdf>

or overview general at

www.travelscoot.com